

PRODUCT DATASHEET

KG255 Drop Rail

Rated to 661lb, the KG255 is a high-strength drop rail designed to support patients in behavioral health en suites and washroom facilities. The sturdy hinged rail can be safely locked in the upright position when not in use to ensure the ligature risk is minimized, while staff can unlock the unit and lower the rail when a patient requires support. The KG255 is manufactured from heavy gauge steel for maximum strength and powder coated for both comfort and to match or contrast the surroundings.



Terms and conditions:

To see our full terms and conditions, please visit:
kingswaygroupglobal.com/en-us/terms

Installation Description

- Place the drop rail unit on the wall and mark the screw hole locations
- Drill holes for the points and insert suitable anchors
- Fix the drop rail in place using the security screws provided

Operation and Maintenance

To use:

- Ensure that you and others are clear of the rail's drop path
- Ensuring the rail is supported, unlock the rail using the supplied key
- Carefully lower the rail into place

After use (raise and lock the rail):

- Ensure a clear path for the rail
- Carefully raise the rail into its upright position, holding it in place
- Lock the rail in its upright position to secure the unit
- You must ensure the rail is correctly locked - pull on the rail to confirm it is locked
- If the rail begins to lower, repeat the locking procedure while supporting the rail

Maintenance

Light lubricant can be used on moving parts and periodic checks should be made to fixings and general smooth operation of the arm.

Cleaning

Should be cleaned as part of regular schedule using mild (non-abrasive) detergents. Avoid acidic cleaners.

Warranty

1 Year Manufacturer's Defect Warranty*


*For faulty manufacture and not for damage

Specification

KG255 Ligature Resistant Drop Rail Support

PRODUCT DIAGRAM

KG255 Drop Rail

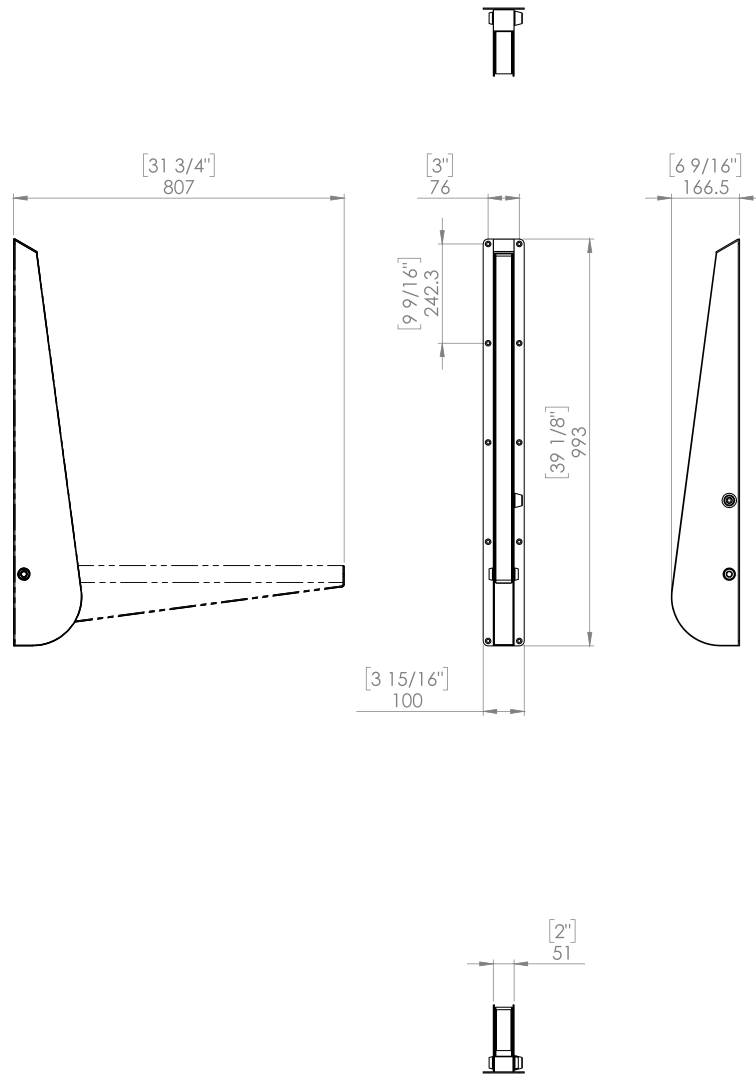

Terms and conditions:

To see our full terms and conditions, please visit: kingswaygroupglobal.com/en-us/terms